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2.1 RECOMMENDATIONS FOR THE USE OF BIOL IN THE TREATMENT OF PROSTATE DISEASES OF VARIOUS ETIOLOGIES

It is recommended to use BIOL medical device in "Therapy" mode for 10 days daily for 120 minutes. It is recommended to use rectal suppositories with the active substance azoximer bromide or bovgyaluronidase azoximer before using the therapy.

Next 10 days take a break and then repeat the course for 10 days.

To check the efficacy of treatment, it is recommended to perform a prostate diagnostic before therapy, and 5-7 days after the repeated course of therapy to make a control diagnosis.

2.2 CONTRAINDICATIONS

Proper examination and diagnosis must be performed, before starting treatment with the device. Please keep up to date with the latest developments and medical publications on devices with low-intensity electromagnetic radiation for detailed information on contraindications and side effects not known at the time of the device's manufacture. Contraindications listed in this section are given at the time of writing of the Instruction. No claims regarding the completeness of this list of contraindications are accepted. Before carrying out the procedures, a medical specialist should be convinced of the expediency of using this procedure, the responsibility for which he bears personally.

The use of the device is contraindicated if a patient has the following signs or pathologies:

- Bleeding, risk of bleeding and blood clotting disorders (haemophilia, haemorrhage, haemorrhoids and ulcers with the risk of bleeding, open wounds and injuries, etc.);
- Severe arterial obstruction (III and IV degree);
- Occlusive vascular diseases, such as obliterating arteriosclerosis and thromboangiitis obliterans (Buerger disease), in which organic occlusion and ischemia are detected;
- Swelling of tissues and the presence of foreign bodies in the affected area;
- Paroxysmal cardiac arrhythmia;
- Epilepsy;
- Gastric ulcer with a complicated course.
- The patient has implantable electronic devices (e.g, heart rate driver (pacemaker), etc.), as well as implantable devices containing metal parts.
- Individual intolerance to the procedure.

2.3 SIDE EFFECTS

Side effects have not been detected. Adverse effects of using the device are possible in case of neglect of the requirements for contraindications.

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